

# FREEDOM SIE AND LOF



## TUBE MEASUREMENT GUIDE

*Determining tube  
length for  
Freedom SIE and  
LOF style hearing  
aid users*



**America Hears® Inc.**  
QUALITY • CONVENIENCE • VALUE

It is necessary to take a measurement of your ear to determine the proper tubing length for your FreedomSIE and LOF instrument.

The Sound Tip needs to fit into the upper portion of the ear canal.

The right ear should be measured with the red side of the measuring tool.

The left ear should be measured with the blue side of the measuring tool.

Place the measuring tool over the ear. It should fit firmly over the ear with little or no gaps as shown.





Position the S,M,L,XL markings by the ear canal.

Looking into the ear canal (you will need some help with this) note which size (S, M, L, XL) tubing you will require.

A medium tube would be required for the example shown.

Once you have determined the size tubing you need, contact America Hears for ordering information.

**Call with any questions:  
800-492-4515**

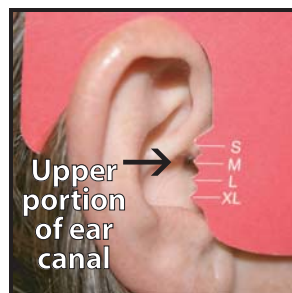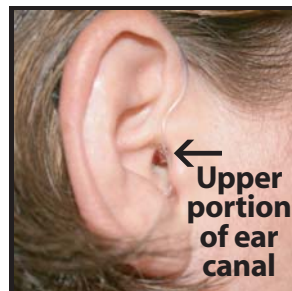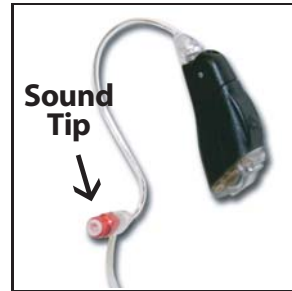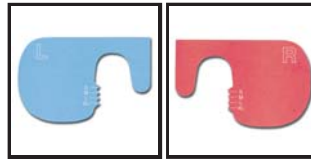

NOTE: It is important that this page is printed to 100% scale. To verify this, measure the printed distance between A and B with a US ruler. The distance should be no more and no less than 3.312". If you find that it is not 3.312", change the "Page Scaling" setting to 100% in the print dialog box and reprint this page .

